

# Care and Maintenance of Fan Units

---

Follow these guidelines to disinfect, clean, and care for your Fan Units (SFVU).



## Disinfectant/cleanser recommendations


- For best results, select a 5.25% bleach disinfecting compound.
- Fan units can also be disinfected and cleaned with compounds containing quaternary ammonium or a solution containing 10%-50% isopropyl alcohol.
- Here are several products that can be used to clean Fan Units:
  - Virox® disinfectant cleaners
  - Clorox® bleach
  - Hepacide Quat® II virucidal disinfectant cleaners
  - PDI® Sani-Cloth® cleaners
  - CaviWipes™ disinfectant cleaners
  - Dispatch® bleach germicidal cleaners

## To maintain your Fan Unit

- Avoid using abrasive pads when cleaning.
- Wipe the surface dry after cleaning.
- If a compound spills, wipe promptly, rinse thoroughly with water, and dry.
- Change Fan Unit HEPA filter every 6 months.  
(See replacement instructions on other side of this sheet.)

# Care and Maintenance of Fan Units

## Replacement Instructions for Fan Units with HEPA Filters

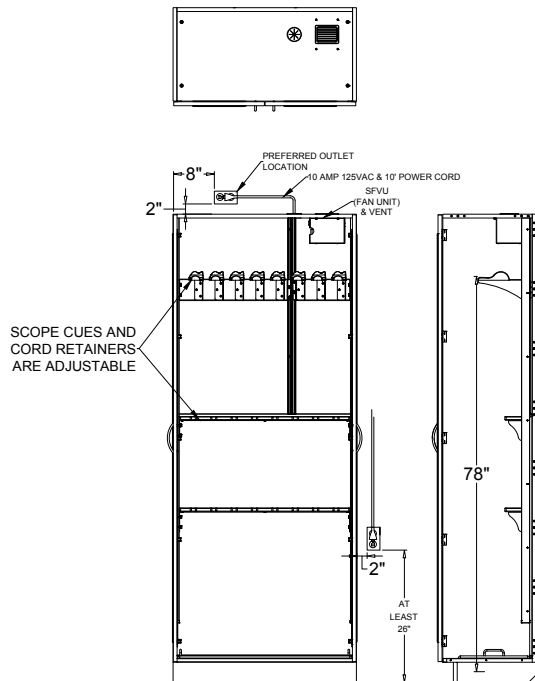

Follow these instructions to replace your HEPA filter.

We recommend you do this **every 6 months for optimal performance**.

- Fan Unit is located at top of cabinet interior.
- Disconnect the power cord from the Fan Unit or wall receptacle.
- Once power is disconnected, hold the sides of the Fan Unit, place thumbs in thumb/finger channel, and lift slightly and pull out filter.
- Install new HEPA filter into Fan Unit; make sure that the gasket side is facing down and filter is pushed all the way to the back of the Fan Unit.
- Reconnect power cord.

