

Follow these guidelines to disinfect, clean, and care for your Fan Units (SFVU).




Disinfectant/cleanser recommendations

- For best results, select a 5.25% bleach disinfecting compound.
- Fan units can also be disinfected and cleaned with compounds containing quaternary ammonium or a solution containing 10%-50% isopropyl alcohol.
- Here are several products that can be used to clean Fan Units:
 - Virox® disinfectant cleaners
 - Clorox® bleach
 - Hepacide Quat® II virucidal disinfectant cleaners
 - PDI® Sani-Cloth® cleaners
 - CaviWipes™ disinfectant cleaners
 - Dispatch® bleach germicidal cleaners

To maintain your Fan Unit

- Avoid using abrasive pads when cleaning.
- Wipe the surface dry after cleaning.
- If a compound spills, wipe promptly, rinse thoroughly with water, and dry.
- Change Fan Unit HEPA filter every 6 months.
(See replacement instructions on other side of this sheet.)

Care and Maintenance of Fan Units

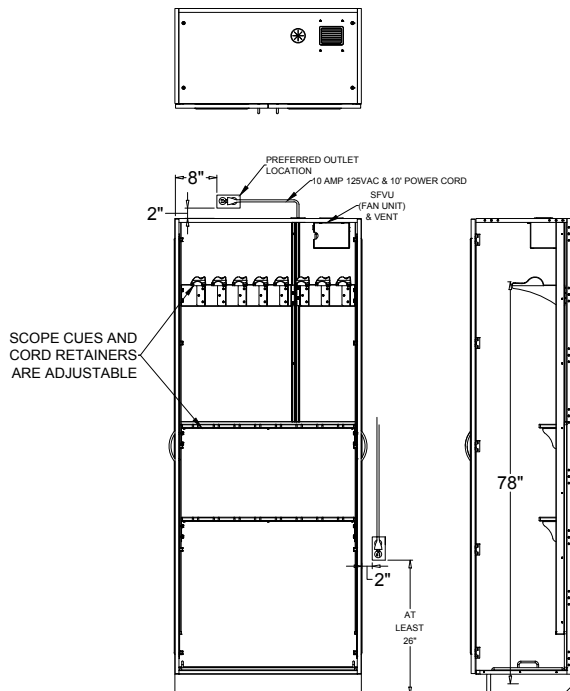

Replacement Instructions for Fan Units with HEPA Filters

Follow these instructions to replace your HEPA filter. We recommend you do this every 6 months for optimal performance.

- Fan Unit is located at top of cabinet interior.
- Disconnect the power cord from the Fan Unit or wall receptacle.
- Once power is disconnected, hold the sides of the Fan Unit, place thumbs in thumb/finger channel, and lift slightly and pull out filter.
- Install new HEPA filter into Fan Unit; make sure that the gasket side is facing down and filter is pushed all the way to the back of the Fan Unit.
- Reconnect power cord.

