

Follow these guidelines to disinfect, clean, and care for your Roam, Tempo, and Rover carts and Evolve cabinets.

## Care and Maintenance of Laminate, AireCore, and Aluminum Composite Surfaces

### Cleaning Materials

- Use compounds that contain quaternary ammonium; they are the best choice for disinfecting and cleaning your cabinets.
- Use a 10%-50% isopropyl alcohol solution as a good option when quaternary ammonium compounds are not available.
- Don't use abrasive cleaning pads.
- Don't leave cleaning compounds on the surface; wipe surfaces dry after cleaning.
- A 10% or less mixture of active bleach can be used but is not recommended.

### Recommended disinfectant/cleaning products

- Hepacide Quat® II virucidal disinfectant cleaners
- PDI® Sani-Cloth® cleaner
- CaviWipes™ disinfectant cleaner
- Virox® disinfectant cleaner

## Care and Maintenance of Stainless Steel Surfaces

### Cleaning Materials

- Use compounds that contain quaternary ammonium; they are the best choice for disinfecting and cleaning your cabinets.
- Use a 10%-50% isopropyl alcohol solution as a good option when quaternary ammonium compounds are not available.
- Use stainless steel polishing products, such as Wieman® and Zep®, to restore surface sheen.
- Don't use compounds containing bleach.
- Don't use abrasive cleaning pads.
- Don't leave cleaning compounds on the stainless steel surface; wipe surfaces dry after cleaning.

### Recommended disinfectant/cleaning products

- Hepacide Quat® II virucidal disinfectant cleaners
- PDI® Sani-Cloth® cleaner (without bleach)
- CaviWipes™ disinfectant cleaner
- Virox® disinfectant cleaner