

Evolve Scope Cabinet

Follow these steps to adjust the location of scope managers and cord retainers in your Evolve scope cabinet.

Tools needed:

- Allen wrench (included)
- Level (optional)



Scope Manager Module



Adjustable Cord Retainer

Adjustable Scope Managers

Step 1

- Use Allen wrench to loosen the 2 screws on either the far left or far right side of the scope manager module. Loosen all 4 screws on 26.75" w cabinet.

Step 2

- Use Allen wrench to loosen the 2 screws on the aluminum track in the middle of the cabinet. (Step 2 not necessary for 26.75" w cabinet)

Step 3

- Slide the module up or down to the desired location; partially tighten screws.

Step 4

- *Recommended:* Use a level to ensure that module height is level across the cabinet.

Step 5

- Use Allen wrench to tighten screws on scope manager module and aluminum track.

Note: 26.75"-wide scope cabinet has 5 managers on module; 36"-wide scope cabinet has 5 managers on left module and 3 on right module; and 40"-wide scope cabinet has 5 managers on left module and 4 on right module.

Adjustable Cord Retainers

Step 1

- To adjust the height of one or both of the cord retainer brackets, use Allen wrench to loosen the 4 screws on the bracket(s).

Step 2

- Slide the bracket(s) up or down to the desired locations; partially tighten screws.

Step 3

- *Recommended:* Use a level to ensure that the bracket height is level across the cabinet.

Step 4

- Use Allen wrench to tighten the screws on the bracket(s).