

Adjust Scope Managers and Cord Retainers in Your Evolve Scope Cabinet

Follow these instructions to adjust the location of scope managers and cord retainers in an Evolve scope cabinet.

Tools needed

- Allen wrench (included)
- Level (optional)



Scope Manager Module



Adjustable Cord Retainer

Adjustable Scope Managers

Step 1

Use Allen wrench to loosen the 2 screws on either the far left or far right side of the scope manager module. Loosen all 4 screws on 26.75" w cabinet.

Step 2

Use Allen wrench to loosen the 2 screws on the aluminum track in the middle of the cabinet. (Step 2 not necessary for 26.75" w cabinet)

Step 3

Slide the module up or down to the desired location; partially tighten screws.

Step 4

Recommended: Use a level to ensure that module height is level across the cabinet.

Step 5

Use Allen wrench to tighten screws on scope manager module and aluminum track.

Note: 26.75"-wide scope cabinet has 5 managers on module; 36"-wide scope cabinet has 5 managers of left module and 3 on right module; and 40"-wide scope cabinet has 5 managers on left module and 4 on right module.

Adjustable Cord Retainers

Step 1

To adjust the height of one or both of the cord retainer brackets, use Allen wrench to loosen the 4 screws on the bracket(s).

Step 2

Slide the bracket(s) up or down to the desired locations; Partially tighten screws.

Step 3

Recommended: Use a level to ensure that bracket height is level across the cabinet.

Step 4

Use Allen wrench to tighten screws on the bracket(s).