

## Roam Scope Cart

Follow these steps to adjust the location of scope managers and cord retainers in your Roam scope cart.

### Tools needed:

- Allen wrench (included)
- Level (optional)



Scope Manager Module



Adjustable Cord Retainer

### Adjustable Scope Managers

#### Step 1

- Use Allen wrench to loosen the 4 screws on the scope manager module.

#### Step 2

- Slide the module up or down to the desired location; partially tighten screws.

#### Step 3

- *Recommended:* Use a level to ensure that module height is level across the cart.

#### Step 4

- Use Allen wrench to tighten screws on scope manager module and aluminum track.

### Adjustable Cord Retainers

#### Step 1

- To adjust the height of one or both of the cord retainer brackets, use Allen wrench to loosen the 4 screws on the bracket(s).

#### Step 2

- Slide the bracket(s) up or down to the desired locations; partially tighten screws.

#### Step 3

- *Recommended:* Use a level to ensure that the bracket height is level across the cart.

#### Step 4

- Use Allen wrench to tighten the screws on the bracket(s).