

Adjust Scope Managers and Cord Retainers in your Roam Scope Cart

Follow these instructions to adjust the location of scope managers and cord retainers in a Roam scope cart.

Tools needed

- Allen wrench (included)
- Level (optional)



Scope Manager Module



Adjustable Cord Retainer

Adjustable Scope Managers

Step 1

Use Allen wrench to loosen the 4 screws on the scope manager module.

Step 2

Slide the module up or down to the desired location; partially tighten screws.

Step 3

Recommended: Use a level to ensure that module height is level across the cart.

Step 4

Use Allen wrench to tighten screws on scope manager module and aluminum track.

Adjustable Cord Retainers

Step 1

To adjust the height of one or both of the cord retainer brackets, use Allen wrench to loosen the 4 screws on the bracket(s).

Step 2

Slide the bracket(s) up or down to the desired locations; Partially tighten screws.

Step 3

Recommended: Use a level to ensure that bracket height is level across the cart.

Step 4

Use Allen wrench to tighten screws on scope manager module and aluminum track.