


Adjusting a Side-Mounted Accessory Rail on a Pace Cart

Follow these instructions to adjust a side-mounted accessory rail on a Pace cart.

Tools needed


- Hex key (included)
- Level (optional)

Attachment hardware included

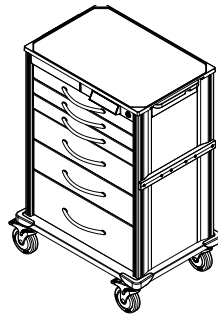


Step 1

Using the hex key, loosen the 2 screws by turning counterclockwise.

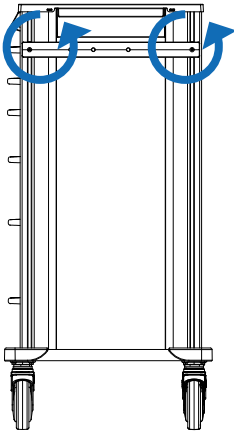

Step 2

Slide the accessory rail to desired location.

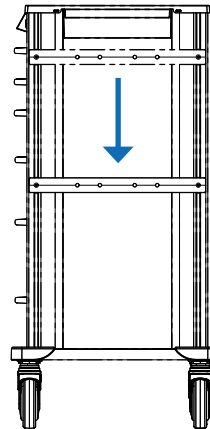

Step 3

Recommended: Use a level to ensure the accessory rail is even.

Step 4

Using the hex key, firmly tighten screws by turning clockwise.

