

Follow these steps to position adjustable cath managers on a metal shelf.

Tools needed:

- #3 Phillips head screwdriver

## Step 1

- Locate the 4 cath manager locking tabs located under the shelf.

## Step 2

- Using a #3 Phillips head screwdriver, partially loosen the 4 lock tab screws.

## Step 3

- Slide the cath manager assembly horizontally to the desired location.

## Step 4

- Tighten the 4 lock tab screws.

Partially loosen the four lock tab screws to slide cath manager assembly to desired location

