

Adjusting Cath Managers

Follow these instructions to position adjustable cath managers on a metal shelf.

Tools needed

- #3 Phillips head screwdriver

Step 1

Locate the 4 cath manager locking tabs located under the shelf.

Step 2

Using a #3 Phillips head screwdriver, partially loosen the 4 lock tab screws.

Step 3

Slide the cath manager assembly horizontally to the desired location.

Step 4

Tighten the 4 lock tab screws.

Partially loosen the four lock tab screws to slide cath manager assembly to desired location

