

# Attaching a Catheter Holder to a Pace Cart

Follow these instructions to attach a catheter holder to a Pace cart.

## Tools needed:


- Hex key (included)
- Level (optional)

Attachment hardware included




## Step 1

Arrange the 4 T-nuts so that they sit on the last thread of the 2 screws on each of the catheter holder mounting rails.

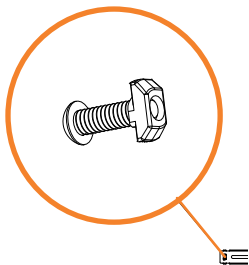

## Step 2

Align the T-nuts vertically, and then push the mounting rails into the 2 side corner channels at the desired height on either the right or left side of the cart.

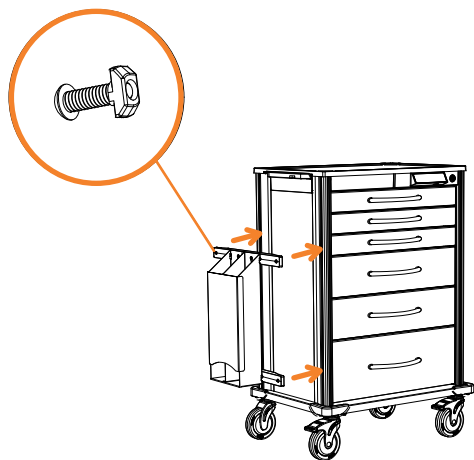

## Step 3

*Recommended:* Use a level to ensure the catheter holder is even.

## Step 4

Using the hex key, firmly tighten the 4 screws on the mounting rails by turning clockwise. If a T-nut isn't tightening, make sure that the T-nut has been placed on the last thread of the screw.

