


Attaching a Cord Wrap to a Pace Cart

Follow these instructions to attach a cord wrap to a Pace cart.

Tools needed


- Hex key (included)
- Level (optional)

Attachment hardware included

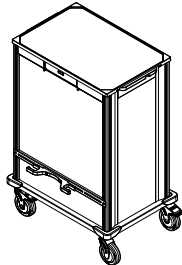


Step 1

Arrange the 2 T-nuts so that they sit on the last thread of the 2 screws on the cord wrap mounting rail.

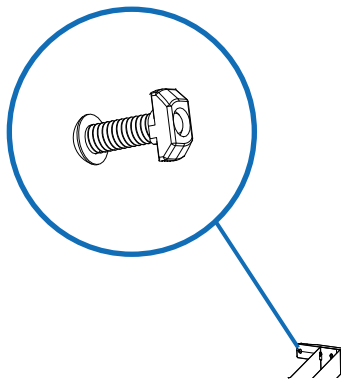

Step 2

Align the T-nuts vertically, and then push the mounting rail with cord wrap into the 2 back corner channels at the desired height.

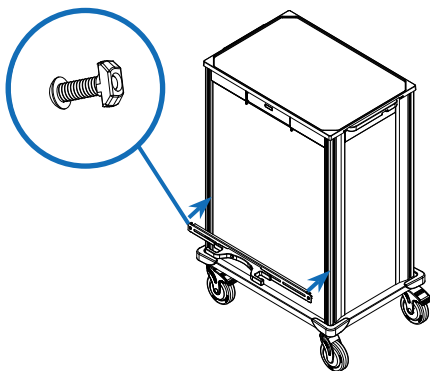

Step 3

Recommended: Use a level to ensure the cord wrap is even.

Step 4

Using the hex key, firmly tighten the 2 screws on the mounting rail by turning clockwise. If a T-nut isn't tightening, make sure that the T-nut has been placed on the last thread of the screw.

