


# Attaching a Task Light to a Pace Cart

Follow these instructions to attach a task light to a Pace cart.

## Tools needed


- Hex key (included)
- Level (optional)

Attachment hardware included

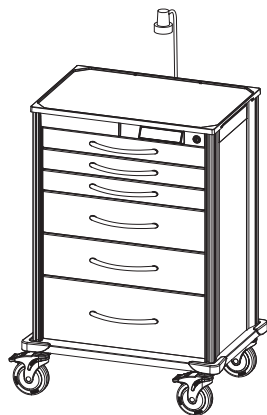


## Step 1

Arrange the 2 T-nuts so that they sit on the last thread of the 2 screws on the task light mounting rail.

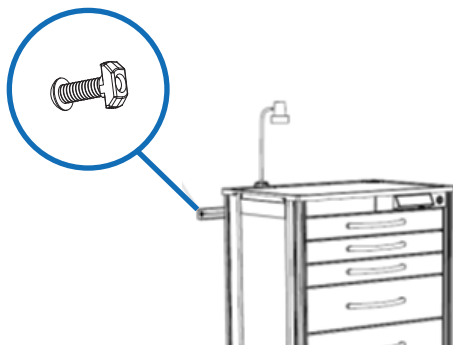

## Step 2

Align the T-nuts vertically, and then push the mounting rail with light into the 2 back corner channels.

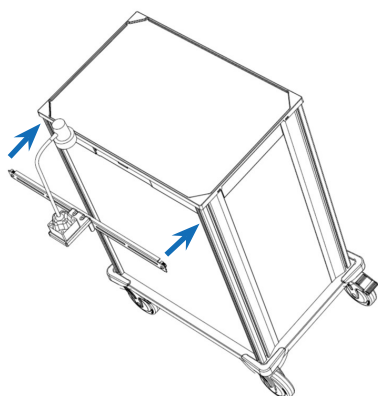


## Step 3

*Recommended:* Use a level to ensure the mounting rail is even.

## Step 4

In order for the battery door to be accessed, position the task light so that there is a minimum of 6" of clearance from the top of the cart to the top of the mounting rail.

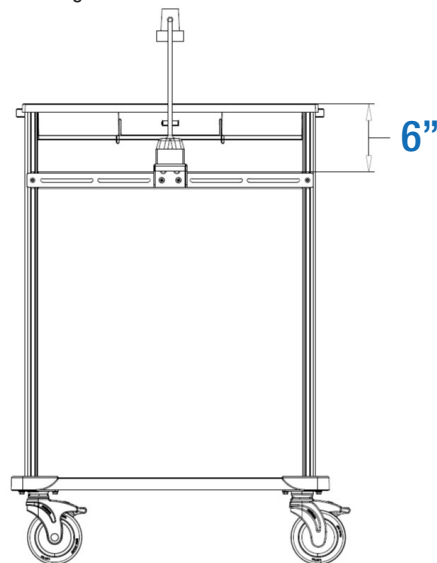

## Step 5

Using the hex key, firmly tighten the 2 screws on the mounting rail by turning clockwise. If a T-nut isn't tightening, make sure that the T-nut has been placed on the last thread of the screw.

