

Attaching a Triple Glove Box Holder to a Pace Cart

Follow these instructions to attach a triple glove box holder to a Pace cart.

Tools needed:


- Hex key (included)
- Level (optional)

Attachment hardware included





Step 1

Arrange the 4 T-nuts so that they sit on the last thread of the 4 screws on each of the glove box holder mounting rails.

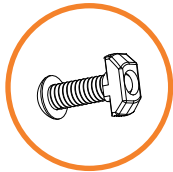

Step 2

Align the T-nuts vertically, and then push the mounting rails with glove box holder into 2 side corner channels at the desired height on either the right or left side of the cart.

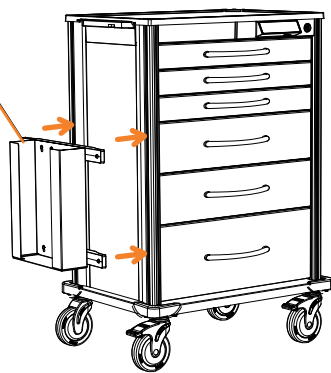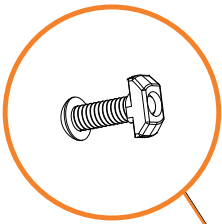

Step 3

Recommended: Use a level to ensure the glove box holder is even.

Step 4

Using the hex key, firmly tighten the 4 screws on the mounting rails by turning clockwise. If a T-nut isn't tightening, make sure that the T-nut has been placed on the last thread of the screw.

