

# Closed Case Cart Accessory Installation Instructions

---




# Closed Case Cart

## Accessory Installation Instructions



Integro Surgical Products

### Adding/ Removing / Changing Shelves


1. Remove thumb screw beneath shelf on left side.

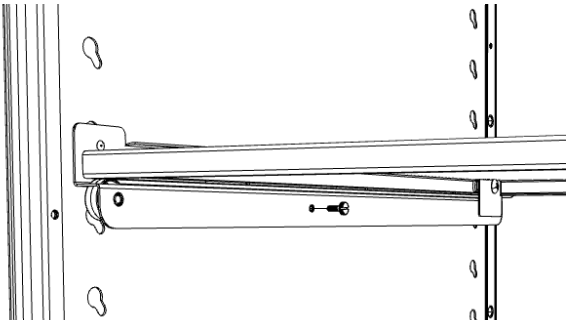


2. For stationary shelves, simply pull out to remove.  
For roll-out shelves, depress lever on right side under shelf and remove shelf.

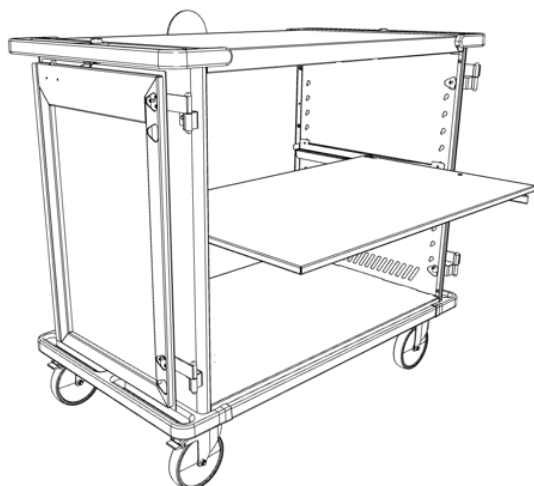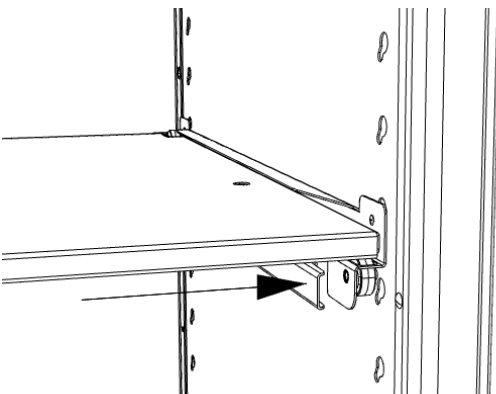


3. Remove shelf slides by pulling slide up and toward you at a 45 degree angle.

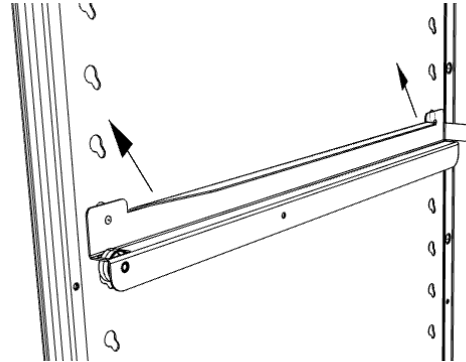

4. Adjust bracket to desired height by sliding it down and away from you at a 45 degree angle and aligning it over the holes in the cart wall.

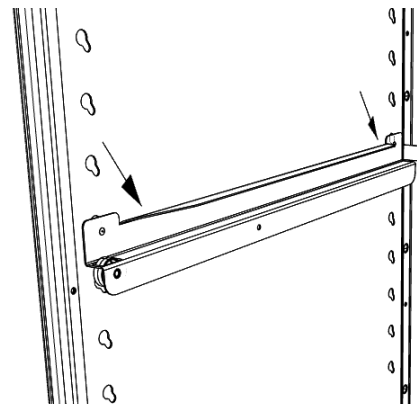

5. Slide shelf into bracket, reinsert thumbscrew, and tighten the screw.

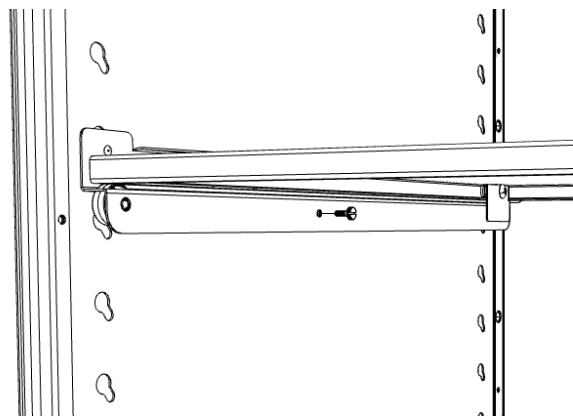


# Closed Case Cart

## Accessory Installation Instructions

Integro Surgical Products

### Adding Clean Dirty Indicator

Use a drill to remove all 5 rivets on the top outer edge of the back wall of the cart as you face it (drill bit provided).

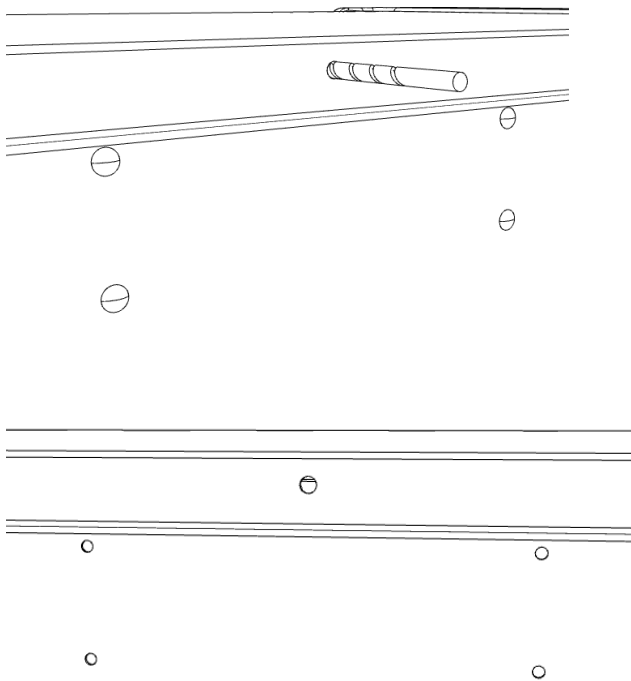

Using provided rivet gun, line up the assembled indicator with the empty rivet holes you just made and rivet to cart.

