

Fan Ventilation Unit

Follow these instructions to install a fan ventilation unit in an Evolve scope cabinet or Roam scope cart.

Tools needed

- Phillips screwdriver

Attachment hardware included.

Fan ventilation unit is mounted in the upper back right corner of the scope cabinet or the upper front right corner of a scope cart.

Step 1

Locate the 4 mounting holes in the top of the cabinet or cart.

Step 2

Attach the fan unit to the cart or cabinet by inserting 4 machine screws and locking washers into the 4 mounting holes.

Step 3

To connect the power cord to the unit, insert the power cord through the grommet on top of the cabinet or cart and plug it into the power connector on the fan unit.

