

Installation Instructions - Evolve Cabinets

Follow these installation instructions for Evolve upper, base, tall, and scope cabinets with AireCore or wood composite construction.

Tools needed

- Flat head screwdriver
- Level

Preparing for cabinet installation

Step 1

Identify the location and placement of your cabinets and clean the floor and surrounding area.

Step 2

If infill panels are needed, attach infill panel directly to the inside of the cabinet with wood screws before installing cabinets.

Installation Instructions - Evolve Cabinets



Installing a single base, tall, or scope cabinet

Step 1

Position base against the wall. Access leveling feet inside of base, and, with a flat head screwdriver, adjust the four leveling feet until the cabinet is level.

Step 2

Mount the base directly to wall studs using appropriate fasteners for the type of wall construction.

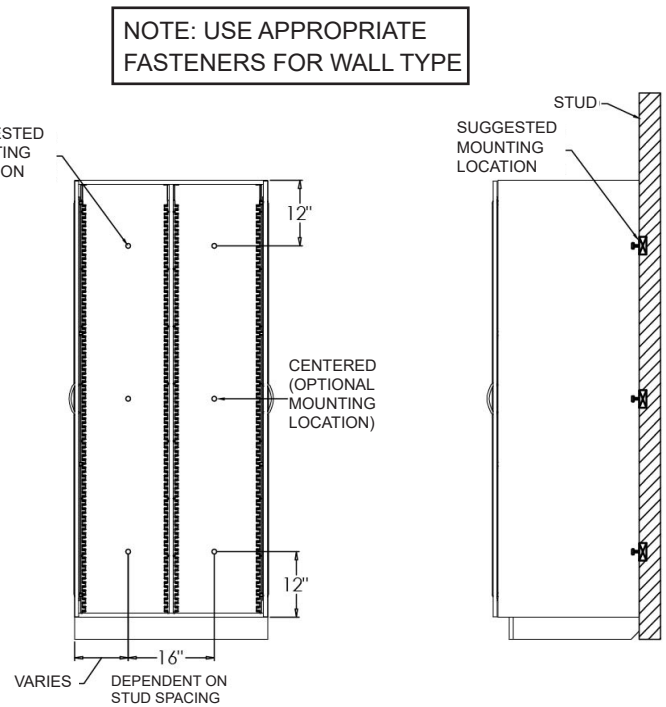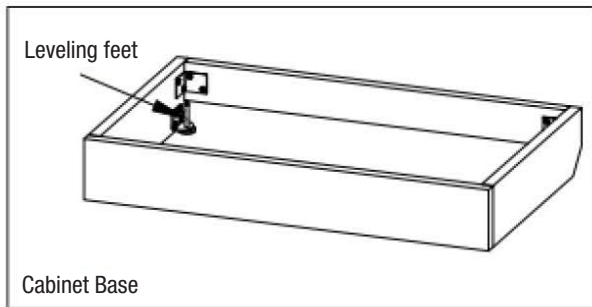


Step 3

Place the cabinet onto the base.

Step 4

Plumb the cabinet and secure it to the wall through the back panel using appropriate fasteners. See figure for placement.

Questions? Call InnerSpace Technical Service representative at 888-435-2256

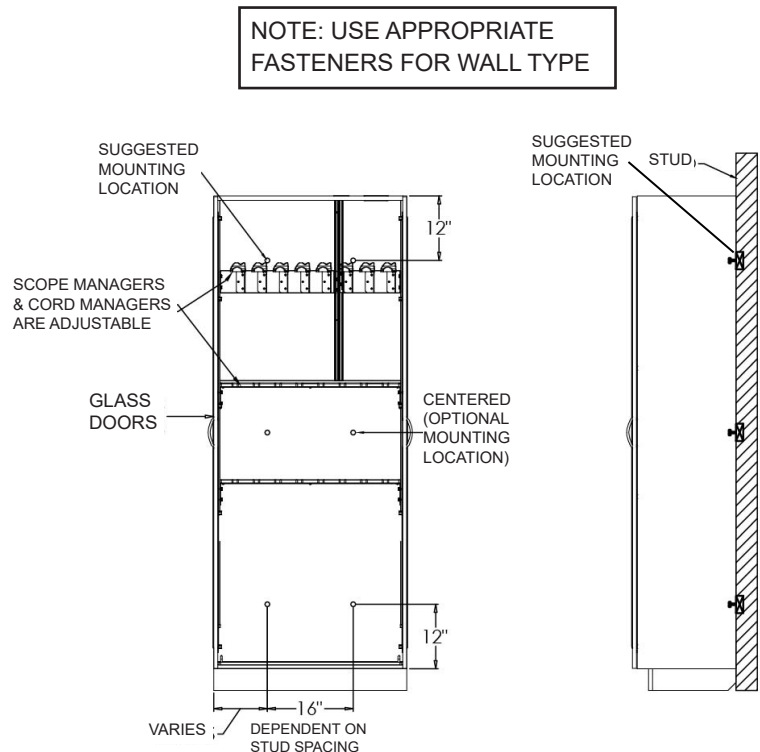

Installation Instructions - Evolve Cabinets


Installing multiple base, tall, and scope cabinets

Step 1


Attach the run of bases together using wood screws, making sure the bases are flush front to back and top to bottom. Position the run of bases against the wall. Access leveling feet inside of bases, and, with a flat head screwdriver, adjust the feet until the run of bases is level.

Step 2

Mount the bases directly to wall using appropriate fasteners for the type of wall construction.

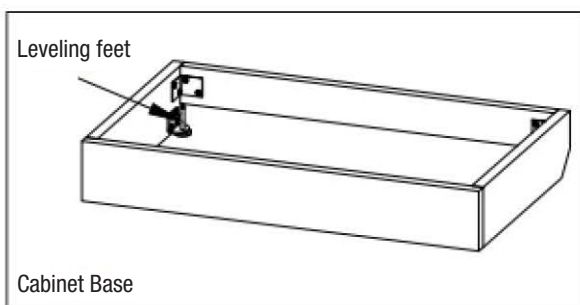

NOTE: USE APPROPRIATE FASTENERS FOR WALL TYPE

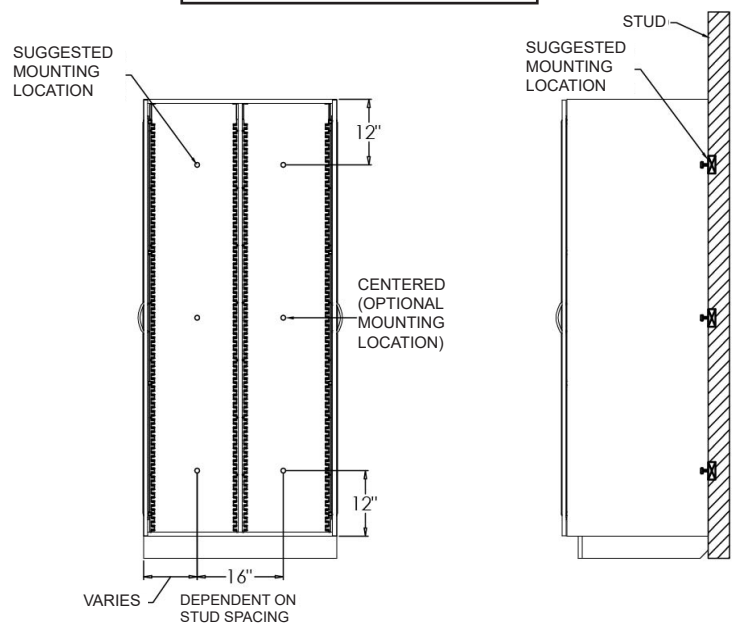

Step 3

Place the end cabinet on the base. For cabinet with no finished ends or two finished ends, center the cabinet. For cabinet with one finished end, flush the adjacent cabinet and base end panels.

Step 4


Plumb the cabinet and secure it to the wall through the back panel using appropriate fasteners. See figure for placement.

Step 5

Repeat steps 1-4 with additional cabinets, making sure each is level and plumb.

Questions? Call InnerSpace Technical Service representative at 888-435-2256

NOTE: USE APPROPRIATE FASTENERS FOR WALL TYPE

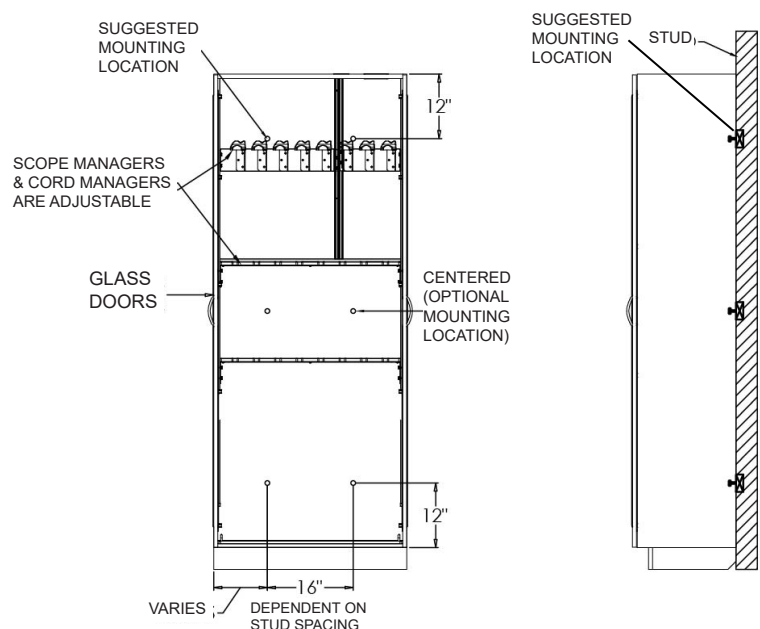

Installation Instructions - Evolve Cabinets

Installing upper wall cabinets

Step 1

Determine location of upper cabinet or cabinet run. Using a level, draw a line marking location of the bottom of the cabinet.


Step 2

Position cabinet bottom at marked line, making sure cabinet is level and plumb, and mount the cabinet directly to wall studs using appropriate fasteners for the type of wall construction. See figure for fastener placement.

Step 3

To eliminate gaps in a cabinet run, drill pilot holes $1\frac{1}{2}$ " from the interior front edge of cabinet and $1\frac{1}{2}$ " from the inside top and bottom of cabinet. Tighten side-by-side cabinets with $1\frac{1}{4}$ " wood screws.

Questions? Call InnerSpace Technical Service representative at 888-435-2256

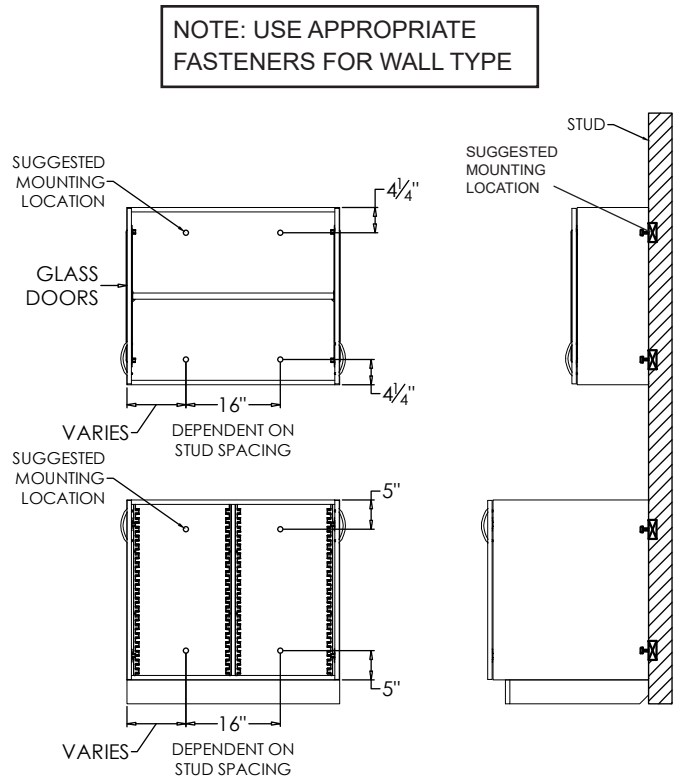


Installation Instructions - Evolve Cabinets

Attaching slope tops to Evolve cabinets

For cabinets with roll-top doors

Step 1

Align the clips on the top of the cabinet and the bottom of the slope top. Place the slope top on the cabinet. Make sure all 4 clips are engaged.

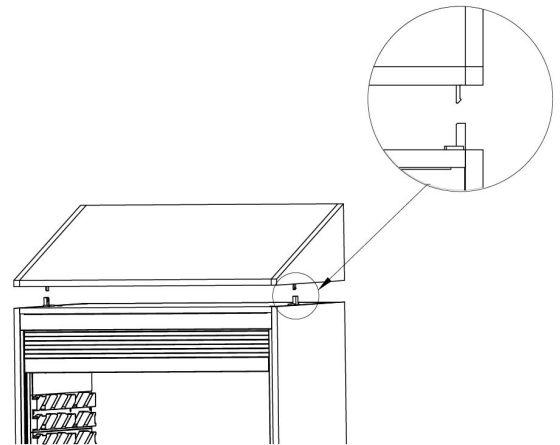

For cabinets with hinged doors

Step 1

Measure the distance from the front edge of the slope top to the center support. See Step 1 figure.

Step 2

Place the slope top on the cabinet. See Step 2 figure.

Step 3

To determine screw placement to connect the slope top to the cabinet, take these measurements and mark locations. See Step 3 and 4 figure.


- 1: screw location from step 1
2. measure 6" toward center from both the right and left side of cabinet
3. measure 1 1/4" from the back of the cabinet

Note: Recommend pre-drilling the marked screw locations.

Step 4

Attach the slope top to the cabinet from the inside of the cabinet using 1 1/4" wood screws (total of 4 screws).

Questions? Call InnerSpace Technical Service representative at 888-435-2256

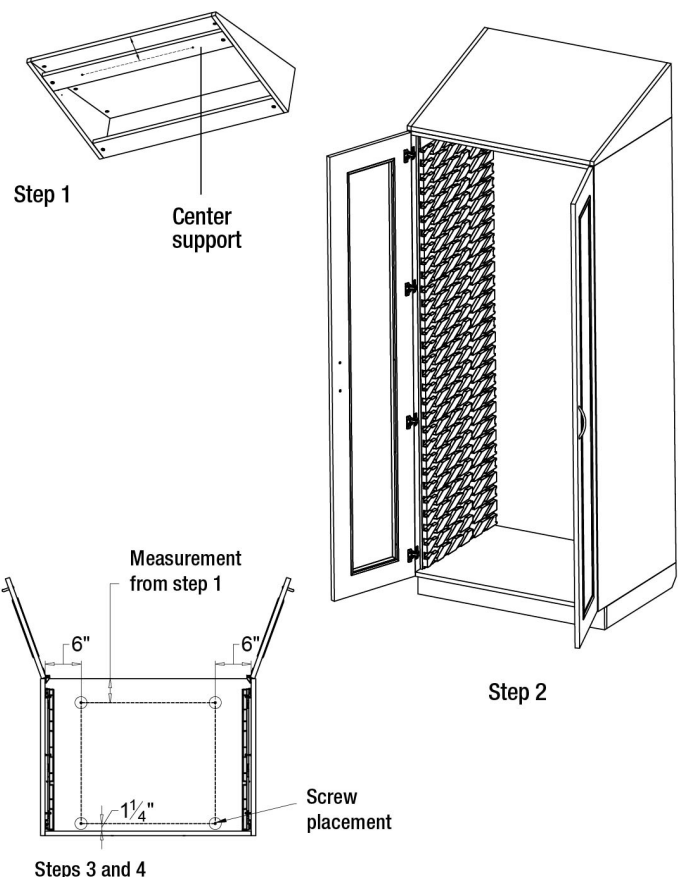



InnerSpace specializes in solutions for all areas and departments within a healthcare facility.

We offer carts, cabinets, open storage, and accessories for supply storage, products with embedded technology for improved efficiency and security, and the Ventaire™ Endoscopy System for the storage, drying, transport, and reprocessing of endoscopes.

InnerSpace is here to help you specify the best products for your needs, whether a standard option or customized solution. We offer complimentary storage analysis and design services. Contact our Customer Service team with your requests or to connect with a sales representative, and find InnerSpace products on GSA Advantage, Premier, and Vizient contracts.



1239 Comstock Street | Marne, Michigan 49435
888-435-2256 phone | 616-743-5944 fax
innerspacehealthcare.com

© 2023 Solaire Medical, Marne, MI 49435
® InnerSpace is a registered trademark of Solaire Medical LLC, Marne, MI 49435
75K0918