Installation Instructions


Evolve Scope Pass-through Cabinet with Floor Anchor Kit

Ventaire[™] Endoscopy System


Scope Drying and Tracking Pass-through Cabinet with Floor Anchor Kit





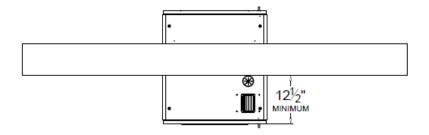

NOTE: USE APPROPRIATE FASTENERS FOR WALL TYPE

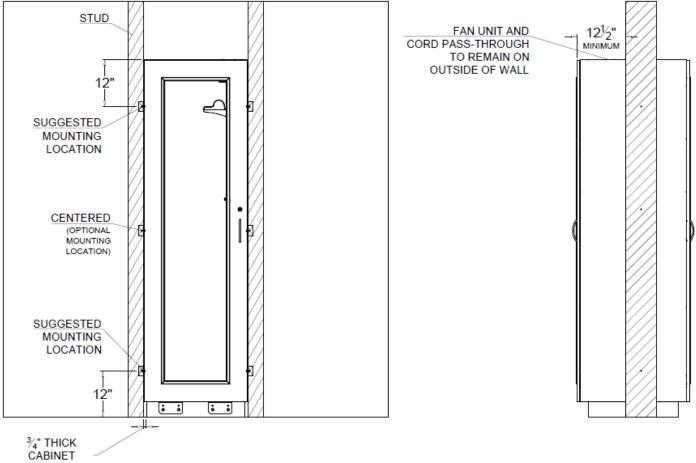



Follow these instructions to install pass-through cabinets.

Tools needed:

- · Flat head screwdriver
- Wood screws for attaching multiple cabinets (if applicable)
- Level
- · Hardware to attach cabinet to wall

Included:

- 4 floor brackets (attached to base)
- 4 cabinet attachment brackets
- Hardware for attaching cabinet to base:
 - 3/8" bolts (8)
 - #8 screws (16)

Caution:

Pass-through cabinets can be unstable when handling, especially when the doors are open. Cabinets are shipped with bracing materials in place that should not be removed until the cabinet is in the location of installation.

Notes:

For ease of installation, we recommend predrilling all attachment locations on the cabinet and base.

Pass-through cabinets come with a fan unit vent and cord pass-through located at the top of the cabinet. These must remain on the outside of the wall and require a minimum 121/2" clearance.

Preparing for cabinet installation

Step 1

Identify the location and placement of your cabinet. Clean floor and surrounding area.


Step 2

If infill panels are needed, attach infill panel directly to the sides of the cabinet with wood screws before installing cabinet.

Installing a single pass-through cabinet


Step 1

Position base into the wall opening. Access leveling feet inside of base. Using a flat head screwdriver, adjust the four leveling feet until the base is level.

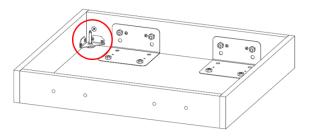

Step 2

Attach base to floor through the 7/16" holes on each floor bracket (2 per bracket, 8 total). Use appropriate fasteners for the type of floor construction.

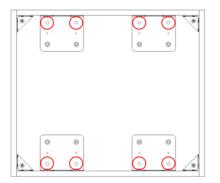

Step 3

Place the cabinet on the base. Remove the bracing materials by removing the 2 attachment screws.






Installation Instructions


Step 4

Line up the cabinet attachment bracket with the predrilled holes on the base. Use the 3/8" bolts to tighten the bracket into place (2 per bracket, 8 total).

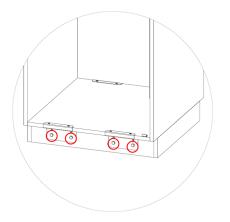

Step 5

Finish attaching the bracket to the cabinet by tightening the #8 screws into place (4 per bracket, 16 total).

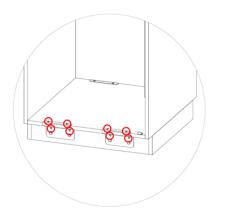


Step 6

Secure the cabinet to the wall through the side panel(s) using appropriate fasteners. Shim where needed. At a minimum, one side panel should be connected to the wall. We recommended attaching to the wall on the same side as the door hinges. See figure for placement.




NOTE: USE APPROPRIATE FASTENERS FOR WALL TYPE

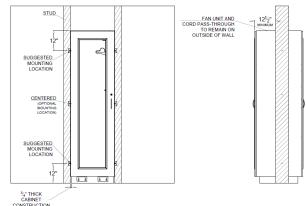

Installing multiple pass-through cabinets


Step 1

Attach the run of bases together using wood screws, making sure the bases are flush front to back and top to bottom. Position the run of bases into the wall opening. Access leveling feet inside of bases (4 per base). Using a flat head screwdriver, adjust the feet until the run of bases is level.

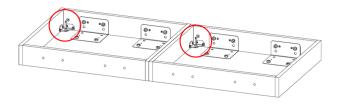

Step 2

Attach base to floor through the 7/16" holes on each floor bracket (2 per bracket, 8 total per base). Use appropriate fasteners for the type of floor construction.

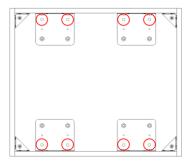



Installation Instructions


Step 3

Place the end cabinet on the base. Remove the bracing materials by removing the 2 attachment screws.




Step 4

Line up the cabinet attachment bracket with the predrilled holes on the base. Use the 3/8" bolts to tighten the bracket into place (2 per bracket, 8 total per base).

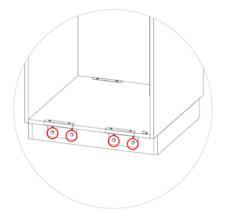

Step 5

Finish attaching the bracket to the cabinet by tightening the #8 screws into place (4 per bracket, 16 total per base).

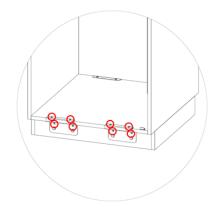

Step 6

Secure the cabinet to the wall through the side panel using appropriate fasteners. See figure for placement.

Step 7

Add additional cabinet, making sure it is level and plumb. Attach to first cabinet through the side panel using wood screws.

Step 8

Repeat with additional cabinets, making sure each is level and plumb.

