

Installing a Push Handle on a Pace Cart

Follow these instructions to attach a push handle to a Pace cart.

Tools needed:


- Hex key (included)
- Level (optional)

Attachment hardware included





Step 1

Arrange the 2 T-nuts so that they sit on the last thread of the 2 screws on the ends of the push handle.

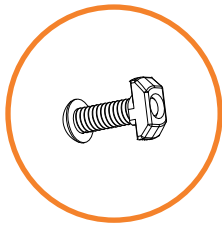

Step 2

Align the T-nuts vertically, and then push the handle into the 2 side corner channels at the desired height on either the right or left side of the cart.

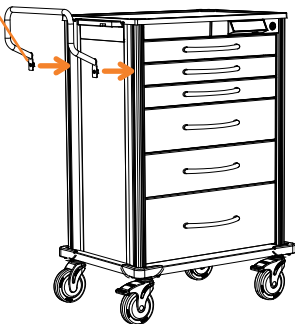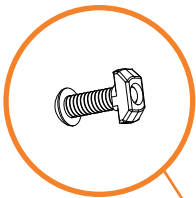

Step 3

Recommended: Use a level to ensure the push handle is even.

Step 4

Using the hex key, firmly tighten the 2 screws by turning clockwise. If a T-nut isn't tightening, make sure that the T-nut has been placed on the last thread of the screw.

