

Follow these instructions to adjust the door on your Roam cart due to misalignment from racking or an uneven floor.

Tools needed:

- 3/16" or 5/32" Allen wrench and 3/8" end wrench for jam nuts

*To lower the door*

**Step 1**

- Locate the socket head cap screw at the bottom front of the cart base. Rotate counterclockwise.

**Step 2**

- Reposition the jam nut on the screw by turning clockwise until it is flush with the head of the socket head cap screw.

*To raise the door*

**Step 1**

- Rotate the jam nut counterclockwise on the socket head cap screw until it is flush with the head of the screw.

**Step 2**

- Rotate the socket head cap screw clockwise.

**Step 3**

- Reposition the jam nut by turning it clockwise until it is flush with the cart base.

*To lower or raise a door on a Roam cart without a jam nut*

*To lower the door*

**Step 1**

- Turn the socket head cap screw counterclockwise.

*To raise the door*

**Step 1**

- Turn the socket head cap screw clockwise.

