

# Roam Door Adjustment

Follow these instructions to adjust the door on a Roam cart due to misalignment from racking or an uneven floor.

## Tools needed

- 3/16" or 5/32" Allen wrench and 3/8" end wrench for jam nuts

## To lower or raise a door on a Roam cart without a jam nut

### To lower the door

#### Step 1

Locate the socket head cap screw at the bottom front of the cart base.  
Rotate counterclockwise.

#### Step 2

Reposition the jam nut on the screw by turning clockwise until it is flush with the head of the socket head cap screw.

### To lower the door

#### Step 1

Turn the socket head cap screw counterclockwise.

### To raise the door

#### Step 1

Turn the socket head cap screw clockwise.

### To raise the door

#### Step 1




Rotate the jam nut counterclockwise on the socket head cap screw until it is flush with the head of the screw.

#### Step 2

Rotate the socket head cap screw clockwise.

#### Step 3

Reposition the jam nut by turning it clockwise until it is flush with the cart base.

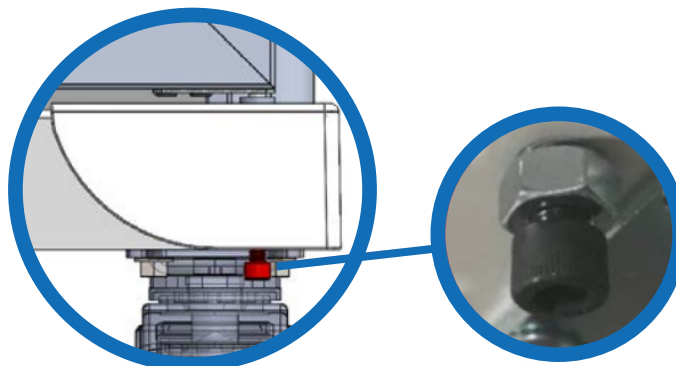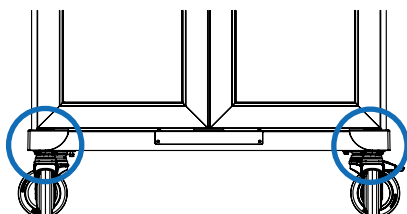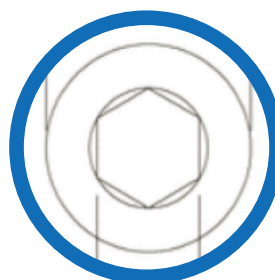

Jam nut