

Follow these instructions to install a scope drying unit inside an Evolve cabinet with adjustable scope managers, cord retainers, and AireCore construction.

Additional installation of filters, electrical components and connections, and training is provided onsite by a product representative. Call your local sales representative or Customer Service at 888-435-2256 for assistance.

#### Tools needed:

- Drill with #2 Phillips bit or a Phillips screwdriver
- 9/64" Allen wrench
- Level

Attachment hardware included

### Step 1

· Loosen, but don't remove, the 8 screws on the scope manager module.




### Step 2

 Slide the scope manager module down the center guide rail to the lowest position in the cabinet.



## Step 3


· Remove the 3 mounting screws on the center guide rail and remove the rail.

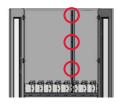



#### Step 4

· Attach the retrofit guide rail to the top 2 holes in the cabinet with 2 screws.

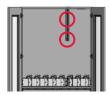

#### Step 5

Slide the scope manager module into place and tighten the 8 screws.



### Step 6

• Use a level to ensure that the module height is even across the cabinet.

Continued on back



Storage solutions for healthcare

Solaire Medical | P.O. Box 2808 | Grand Rapids, Michigan 49501 888-435-2256 phone | 616-743-5944 fax | solairemedical.com



### Step 7

 Remove the 5 screws from the underside of the cord retainer bracket and remove the cord retainer.






### Step 8

Attach the new cord retainer to the bracket using the 5 screws.


#### Step 9

 Position the pump 1½" from the upper left corner and 5/8" from the top of cabinet and attach using 4 wood screws.

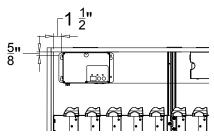

#### Step 10

 Position the manifold a minimum of 14" from the bottom of the pump to the top of the manifold and attach to the 4 corners of the manifold with 4 wood screws.

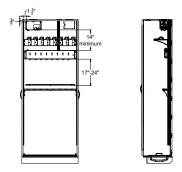

# Step 11

 Position the tubing management strip 17"-24" from the bottom of the top manifold to the tubing management strip. Attach the tubing management strip using 3 wood screws.

