

Switching the Location of the Pull-out Work Surface on Narrow Tempo Carts

Follow these steps to switch the side that the pull-out work surface extends from.

Tools needed:

- Phillips head screwdriver

Step 1

- Open top drawer halfway.

Step 2

- Locate drawer glide lock tabs on each side of drawer; swing tabs to release and remove drawer.

Step 3

- Use Phillips head screwdriver to remove screw and spacer from Hole #1.

Step 4

- Pull work surface through to the opposite side.

Step 5

- Insert screw and spacer in Hole #2 to lock in place.

