

# Attaching an IV Pole to a Tempo Cart

Follow these instructions to attach an IV pole to a Tempo cart.

## Step 1


- Insert the top section of IV pole into the bottom section and rotate top section clockwise until tightened.

## Step 2

- Insert IV pole into the hole on the back of cart surface.

## Step 3

- Rotate pole clockwise until tightened.

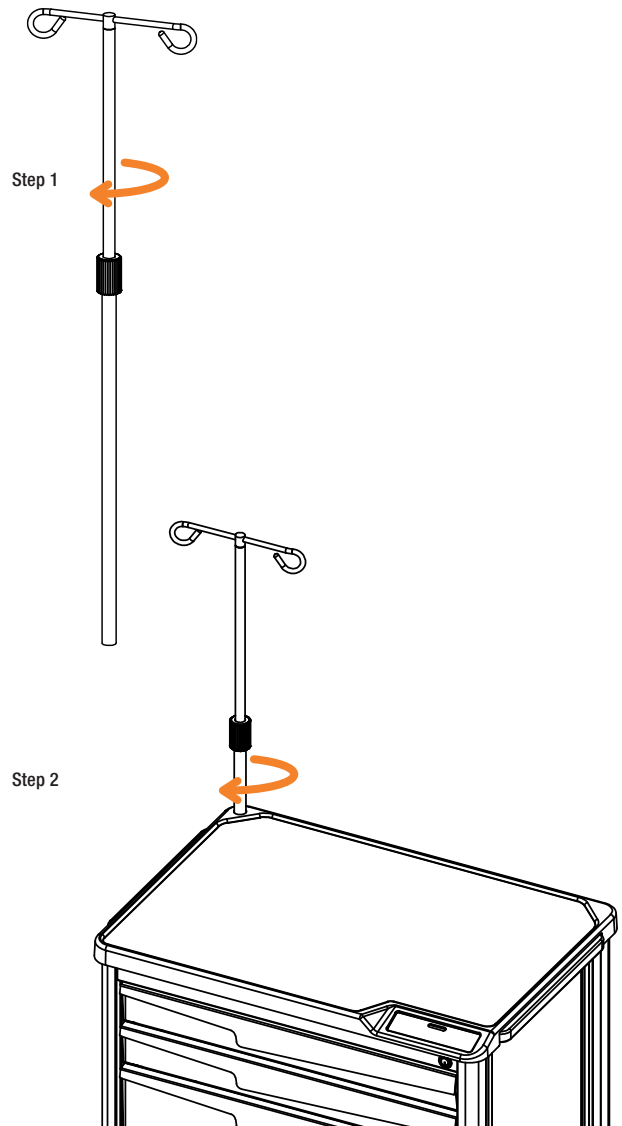



InnerSpace® specializes in storage solutions for all areas of your healthcare facilities. We offer a cohesive system of carts, cabinets, and modular casework that improves staff and facility efficiency and an organization's bottom line. Our InnerSpace portfolio is supplemented with the Quick line of open storage products.

You'll find InnerSpace's full product offering on GSA Advantage, Premier, and Vizion contracts.

Contact our service team with your requests or to connect with a sales representative. We're here to help you specify the best storage products for your needs, whether a standard option or customized solution. We offer complimentary storage analysis, design services, CAD support, and logistics and installation assistance.



1239 Comstock Street | Marne, Michigan 49435

888-435-2256 phone | 616-743-5944 fax | [innerspacehealthcare.com](http://innerspacehealthcare.com)

© 2020 Solaire Medical, Marne, MI 49435  
© InnerSpace is a registered trademark of Solaire Medical, Marne, MI 49435  
0420



QUICK