

Attaching a Task Light to a Tempo Cart

Follow these instructions to attach a task light to a Tempo cart.

Tools needed:




- #2 Phillips head screwdriver
- Attachment screws included

Step 1

- Line up the 2 holes on the task light bracket with the 2 holes under the right back corner of cart surface.

Step 2

- Secure task light to the cart surface using the 2 included screws.

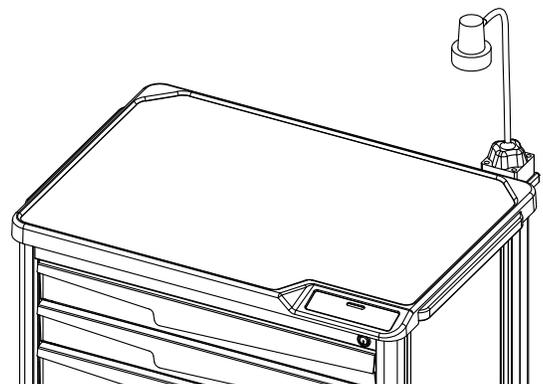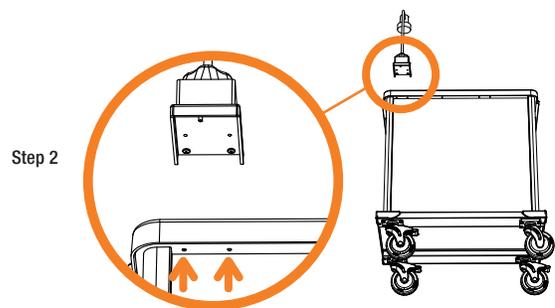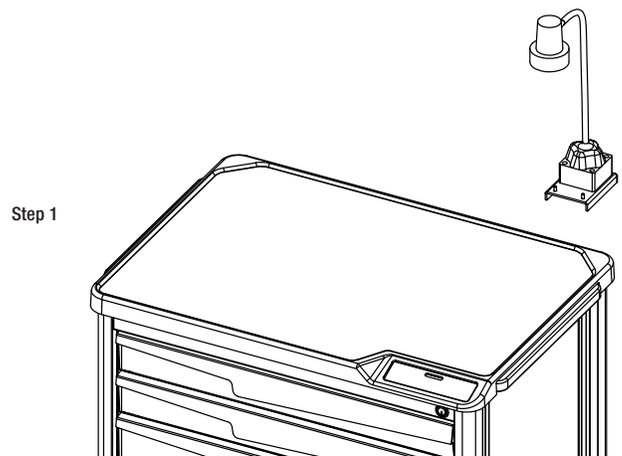

InnerSpace

A SOLAIRE MEDICAL COMPANY

InnerSpace® specializes in storage solutions for all areas of your healthcare facilities. We offer a cohesive system of carts, cabinets, and modular casework that improves staff and facility efficiency and an organization's bottom line. Our InnerSpace portfolio is supplemented with the Quick line of open storage products.

You'll find InnerSpace's full product offering on GSA Advantage, Premier, and Vizion contracts.

Contact our service team with your requests or to connect with a sales representative. We're here to help you specify the best storage products for your needs, whether a standard option or customized solution. We offer complimentary storage analysis, design services, CAD support, and logistics and installation assistance.

solaire
MEDICAL

P.O. Box 2808 | Grand Rapids, Michigan 49501
888-435-2256 phone | 616-743-5944 fax | solairemedical.com



QUICK!