



Follow these instructions to attach a connector to a scope.

## Kit includes:

- Connector
- Biopsy port cover
- Biopsy port connector
- Auxiliary water connector

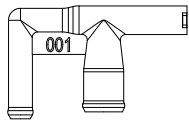

Connector

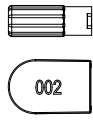

Biopsy port cover




Biopsy port connector



Auxiliary water connector


## Step 1

Attach the connector to the manifold tube

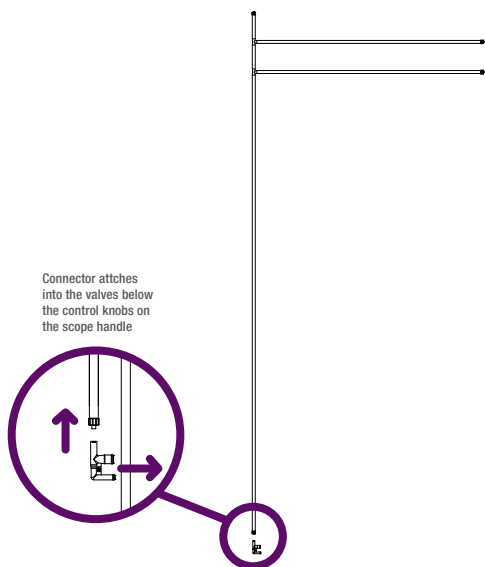

## Step 2


Insert the connector into the suction and air water valve ports below the control knobs on the scope body

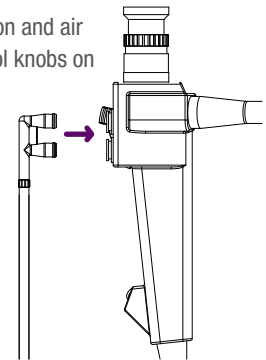

## Step 3

Attach the auxiliary water connector to the manifold auxiliary water tube, and then screw the connector into the water jet port on the scope


NOTE: Thread only until met with resistance; overtightening can strip the connector thread on the auxiliary water jet port

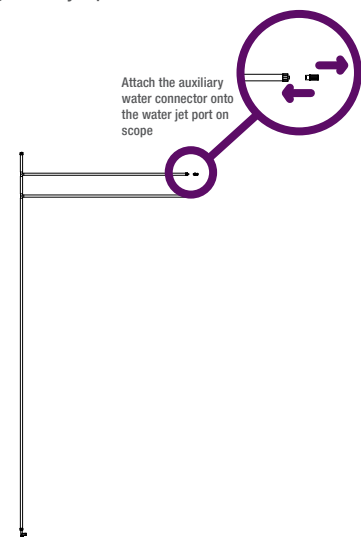

# Connector Kit 12 (SVCK12)

## Step 4

Slide the biopsy port connector onto the biopsy valve on the right side and screw the biopsy tube to the connector

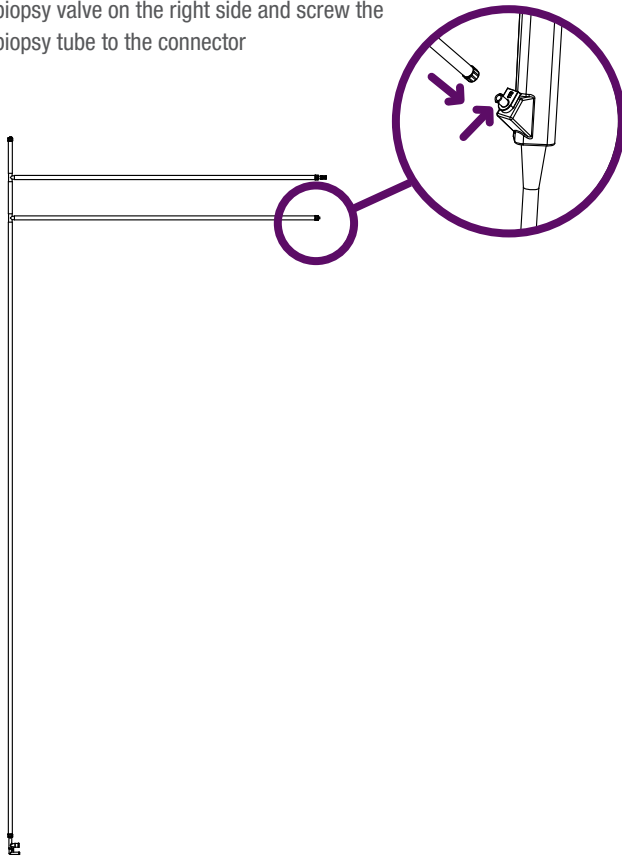

## Step 5

Slide the cover onto the biopsy valve on the left side

