


Connector Kit Instructions

Connector Kit 21 (SVCK21)

Follow these instructions to attach a connector to a scope.


Kit includes:

- Connector
- Biopsy port cover
- Biopsy port connector
- Luer coupler

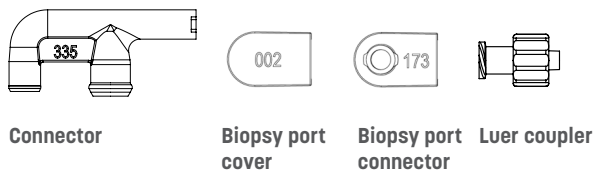


Step 1

Attach the connector to the manifold tube

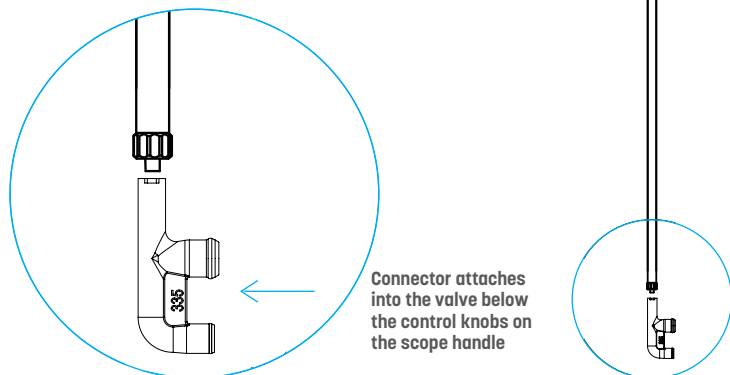


Step 2

Insert the connector into the suction and air water valve ports below the control knobs on the scope body

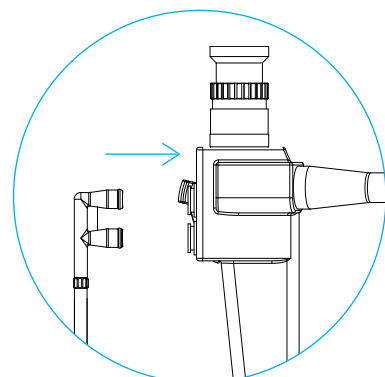

Step 3

Attach the luer coupler to the manifold auxiliary water tube, and then screw the coupler into the water jet port of scope (if applicable)

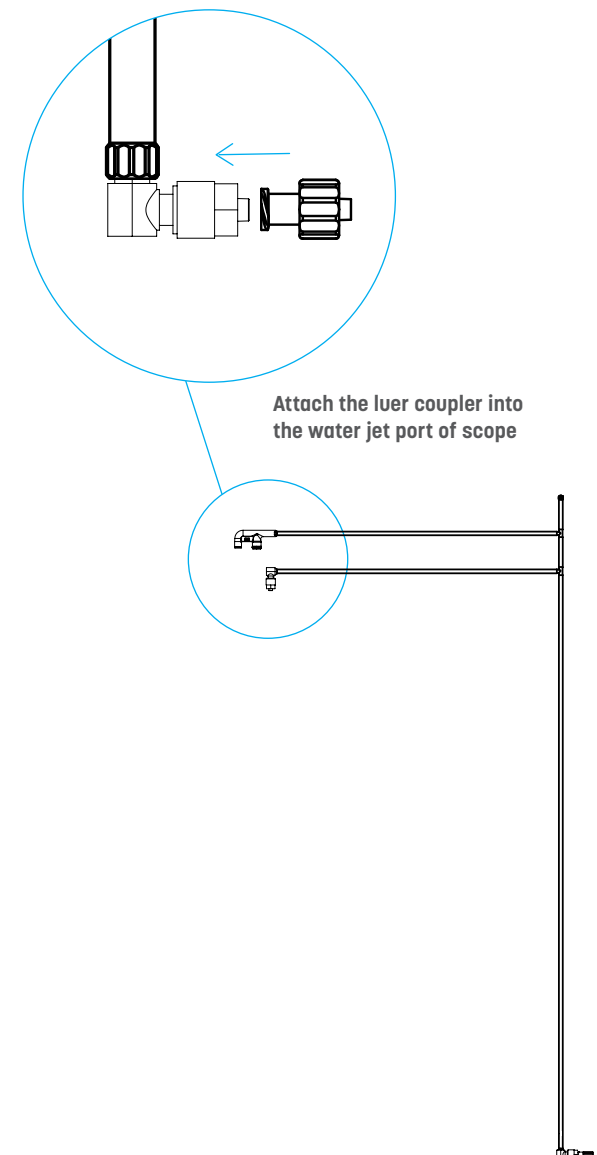


Connector Kit Instructions

Connector Kit 21 (SVCK21) continued

Step 4

Slide the cover onto the biopsy valve on the left side

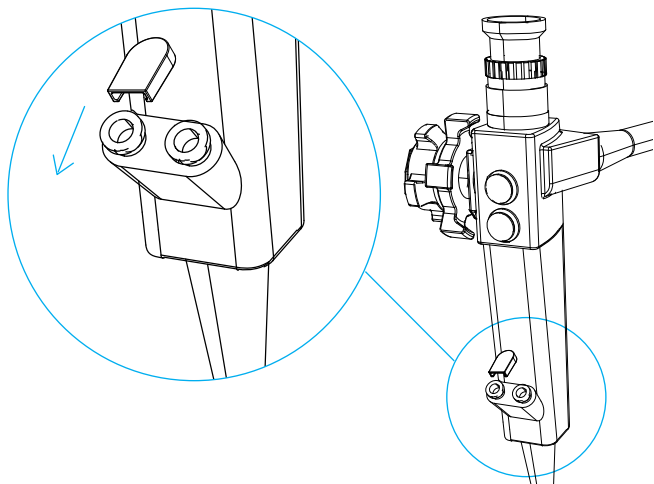

Step 5

Slide the biopsy port connector onto the biopsy valve on the right side and screw the biopsy tube to the connector

