


Connector Kit 6 (SVCK6)

Follow these instructions to attach a connector to a scope.


Kit includes:

- Connector
- 2 biopsy port covers
- Auxiliary water connector

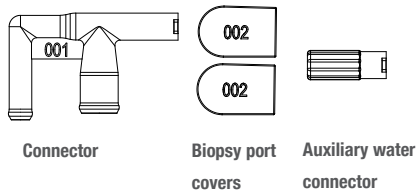


Step 1

Attach the connector to the manifold tube

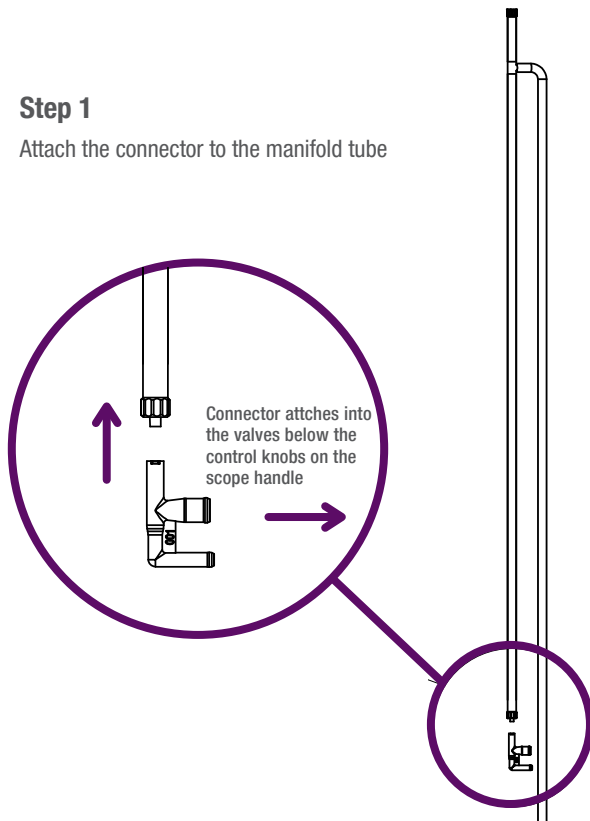

Step 2


Insert the connector into the suction and air water valve ports below the control knobs on the scope body

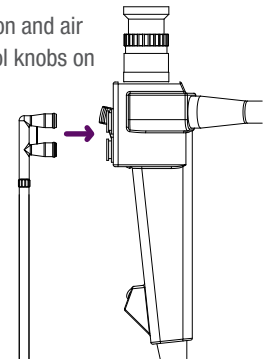

Step 3

Attach the auxiliary water connector to the manifold auxiliary water tube, and then screw the connector into the water jet port on the scope

NOTE: Thread only until met with resistance; overtightening can strip the connector thread on the auxiliary water jet port

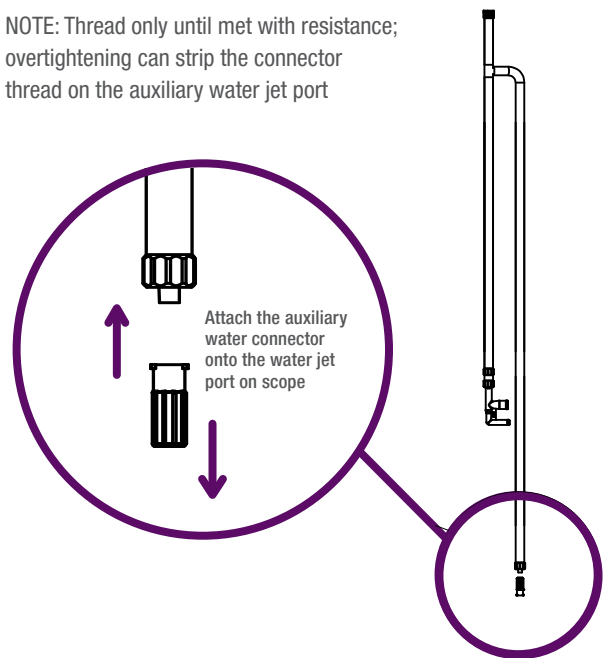

Connector Kit 6 (SVCK6)

Step 4

Slide the covers onto the 2 biopsy valves

