


Connector Kit 7 (SVCK7)

Follow these instructions to attach a connector to a scope.

Kit includes:

- Connector
- Luer plug
- Auxiliary water connector

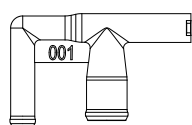

Connector




Luer plug



Auxiliary water connector


Step 1

Attach the connector to the manifold tube

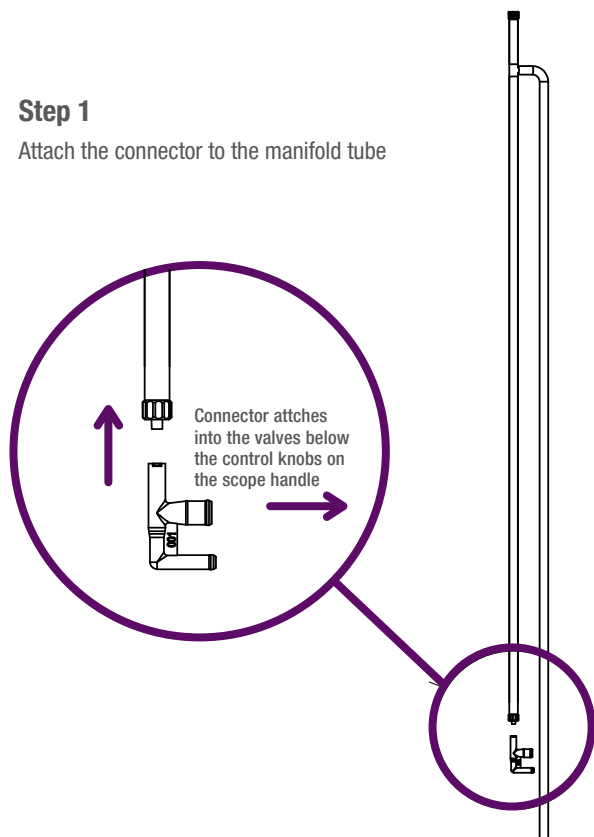

Step 2


Insert the connector into the suction and air water valve ports below the control knobs on the scope body

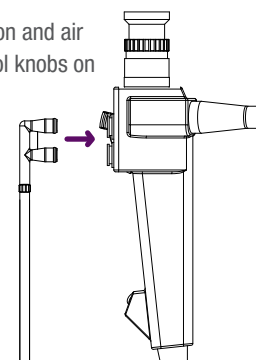

Step 3

Attach the auxiliary water connector to the manifold auxiliary water tube, and then screw the connector into the water jet port on the scope

NOTE: Thread only until met with resistance; overtightening can strip the connector thread on the auxiliary water jet port

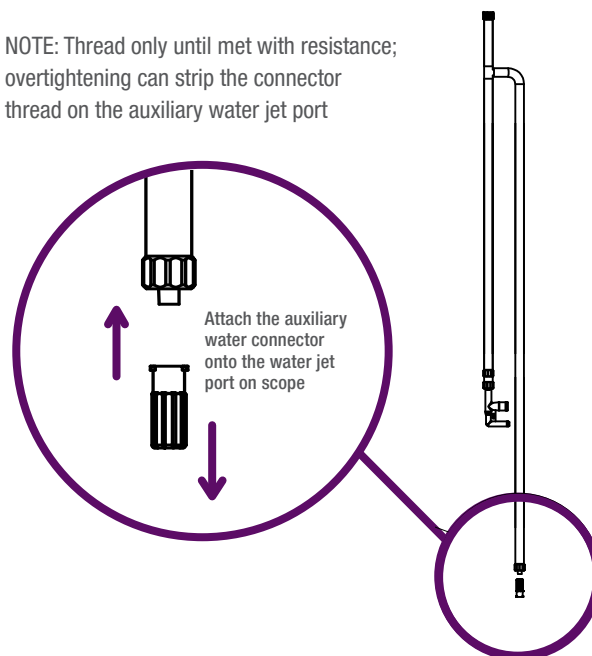

Connector Kit 7 (SVCK7)

Step 4

Screw the plug into the biopsy valve

