


# Connector Kit 9 (SVCK9)

Follow these instructions to attach a connector to a scope.

## Kit includes:

- Connector
- Luer plug
- Auxiliary water connector

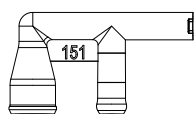

Connector




Luer plug



Auxiliary water connector


## Step 2

Insert the connector into the suction and air water valve ports below the control knobs on the scope body

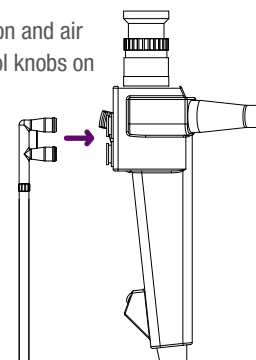

## Step 1

Attach the connector to the manifold tube

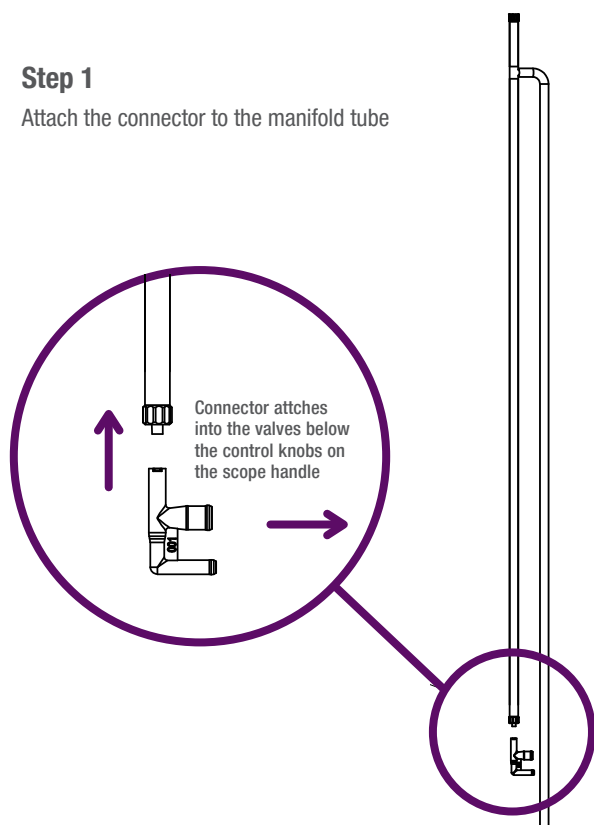


Connector attaches into the valves below the control knobs on the scope handle

## Step 3

Attach the auxiliary water connector to the manifold auxiliary water tube, and then screw the connector into the water jet port on the scope

NOTE: Thread only until met with resistance; overtightening can strip the connector thread on the auxiliary water jet port

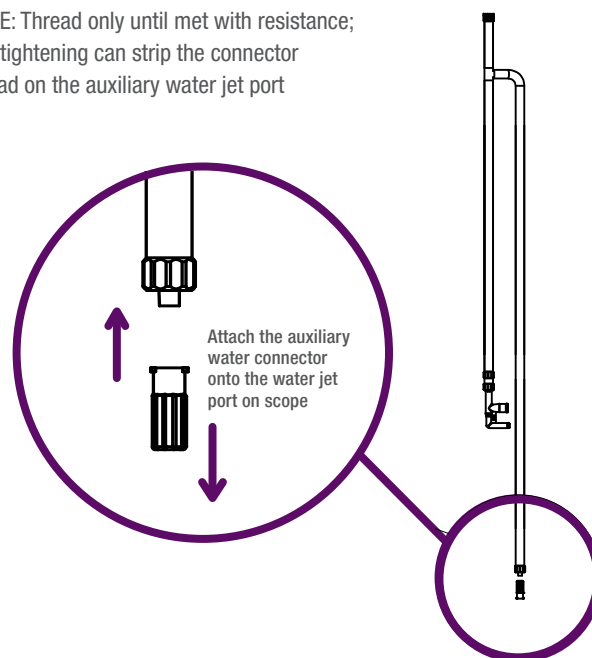

Attach the auxiliary water connector onto the water jet port on scope

## Step 4

Screw the plug into the biopsy valve

