

Instructions for Use

Downloading USB Data



Figure 1: Single Chamber Header Control

Insert Flash Drive

Follow these instructions to download data from your warming cabinet.

1. Insert USB flash drive into USB slot under data symbol (📁) as shown above (Figure 1).
2. Data download will automatically begin.
3. During download display will read:
5. Data is saved on the flash drive as a .CSV file for easy import into a spreadsheet.



4. Once the display shows the temperature again, it is safe to remove the flash drive.
6. Data is secure (not reported or connected to any network).

Sample Report

SM263036SWCGR2B Single, S/N: 062112-D01, 031115

Date/Time	Temperature	°C/°F
3/10/2015 23:12	106°	F
3/10/2015 23:43	106°	F
3/11/2015 0:12	106°	F
3/11/2015 0:43	106°	F
3/11/2015 1:12	106°	F