

Integro Surgical Products

Instruction Guide for Linen Hampers



Read the instructions and safety information in this manual before operating this product.

Table of Contents

| | |
|----------------------------|---|
| Features..... | 3 |
| Care and Maintenance | 4 |

Unpacking and Inspection

InnerSpace linen hampers are heavy. We recommended that two people assist in the unpacking.

- If the shipping container appears to be damaged in any way, contact the shipping company immediately. Save all damaged packaging materials.
- Carefully inspect the linen hamper while unpacking. If any damaged or defective parts are seen, call InnerSpace Customer Service (888-435-2256).
- Linen hampers are shipped fully assembled and ready for immediate use. No assembly is required.



Standard

- Fully welded stainless steel construction
- Stainless steel platform to support bags and for ease of removal
- 2" swivel casters

Optional

- Foot-operated stainless steel lid

| Product Number | Overall Dimensions | Diameter | Foot-operated Lids |
|----------------|--------------------------|----------|--------------------|
| SMLH18 | 35"H x 57 1/4" Perimeter | 18" | N |
| SMLH18FOL | 35"H x 57 1/4" Perimeter | 18" | Y |
| SMLH25 | 35"H x 78 5/8" Perimeter | 25" | N |
| SMLH25FOL | 35"H x 78 5/8" Perimeter | 25" | Y |

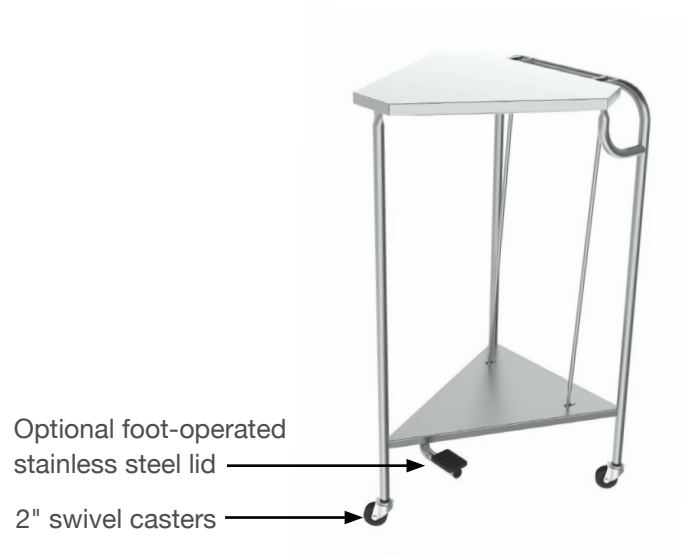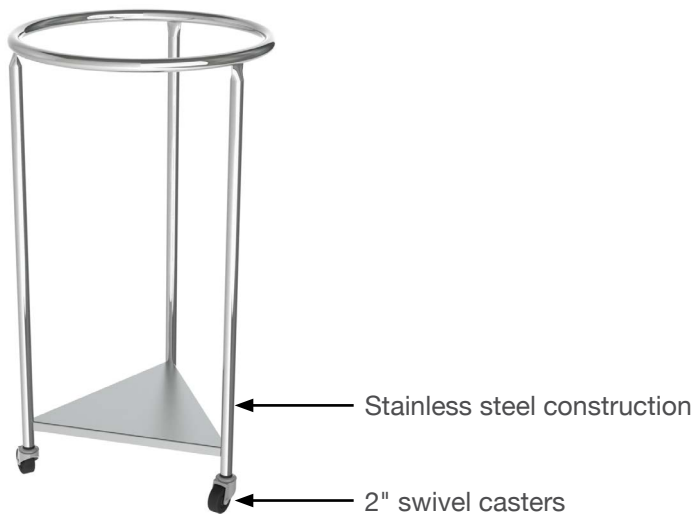

Stainless steel linen hampers must be cleaned on a regular basis in order to prevent unnecessary damage to the stainless steel surface. When cleaning the stainless steel linen hampers, make sure to use the proper cleaning agents and cleaning materials.

Do not use the following materials on the stainless steel surfaces of the linen hampers:

- Abrasive pads
- Scrapers
- Steel wool
- Wire brushes

Do not use the following agents on the stainless steel surfaces of linen hampers:

- Hard water
- Bleach or any products containing chlorine
- Hydrochloric acid
- Denatured alcohol

Always follow the manufacturer's directions for any chemicals used on the stainless steel surfaces of the linen hampers.


Use the following cleaning materials and agents:

- Soft, clean lint-free cloth
- Distilled water
- Distilled water with mild detergent
- Cleaners approved for use on stainless steel

Cleaning a Stainless Steel Linen Hamper

Stainless steel:

- Use commercially prepared stainless steel cleaner and follow the manufacturer's directions.
- Clean the stainless steel linen hampers with the grain of the panel.

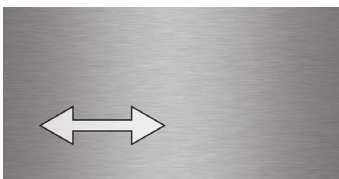

Decals or printed labels:

- Use distilled water and a mild detergent applied with a lint-free cloth. Do not use any chemical cleaners.

Users of the linen hamper are responsible for the proper care, use, and maintenance of the equipment. The linen hamper can become damaged and unsafe if used improperly.

Linen Hampers

- Never overload the linen hamper.
- Do not drop heavy loads on the linen hamper.
- Do not use the linen hamper at high speeds.

Using the linen hamper in any of the above conditions can result in failure of the casters, wheels, or other parts of the equipment.

Periodically inspect the bottom of the linen hamper and look for any of the following:

- Broken welds.
- Loose nuts or bolts. These should be tightened.
- Frame distortion that could be caused by an overloaded hamper or by impacts with other objects while moving the hamper.

Casters and Wheels

- Make sure the casters are in good condition. If a swivel caster has excessive play due to wear, it may need to be replaced. If casters don't turn freely, corrosion or dirt may require their replacement.
- Wheels should be checked for visible tread wear. Flat spots due to accumulation of material, such as string or thread, can cause the wheel to bind. The axle bolt nut should be removed and the foreign material removed. If the internal components are intact, the caster can be reassembled.
- Rubber tires can be badly worn and cause erratic steering, bumping, load shifting, and floor damage.