Scope Drying and Tracking Retrofit Unit



Support scope drying and storage processes, safety protocols, and reduce the chances (and time and expense) of reprocessing expired endoscopes with the scope drying and tracking system. The system meets the 2021 Multisociety Task Force Scope Storage and Drying Guidelines.

The system continuously filters air into internal scope channels to efficiently and thoroughly dry scopes. The easy-to-operate system features timed drying and venting cycles. HEPA filtered air removes 99.9% retention rate of particles less than $0.1\mu m$. It works with all major endoscope brands.

- Continuous venting of HEPA filtered air through endoscope channels, providing safe scope storage 24/7
- 10-minute default drying cycle
- . Modify or add to scope inventory at any time
- · Customizable report captures all scope information and status
- Monitor scope storage from touchscreen on cabinet or remotely via the InnerSpace Cloud platform



Scope Drying and Tracking Retrofit Unit





Scope Drying and Tracking Retrofit Unit

Dimensions

Compressor: 9.61"w x 3.56"d x 5.96"h

5- and 10-port manifold: 24.30"w x 0.64"d x 4.88"h 8- and 16-port manifold: 33.62"w x 0.64"d x 4.88"h 9- and 18-port manifold: 37.6"w x 0.64"d x 4.88"h

Interior Configuration:

26.75"w cabinet: holds a 5- or 10-port manifold 36"w cabinet: holds a 8- or 16-port manifold 40"w cabinet: holds a 9- or 18-port manifold

Options

InnerSpace Cloud Services for remote monitoring of scopes

Specifications

Compressor air output:

• up to 45 LPM

Compressor decibels:

· 62dB, with doors closed


Electrical requirements for tracking unit:

- 120 VAC, 60Hz, 15-amp standard duplex receptacle
- Includes a 125V, 10-amp, 10-foot-long power cord



Scope Drying and Tracking Retrofit Unit

Product Number Reference

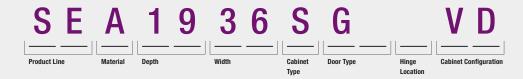

Product Numbers

To order a scope drying and tracking retrofit unit, first specify the replacement door size to match the cabinet size, and then specify the drying and tracking retrofit unit.

Door for 26.75"w x 48"h left-hinged cabinet

AireCore: SEA1927SGLWVD (glass door), SEA1927SSLWVD (solid door)

AireCore with Brushed Aluminum: SEB1927SGLWVD (glass door), SEB1927SSLWVD (solid door)

Stainless steel: SES1927SGLWVD (glass door), SES1927SSLWVD (solid door)

Door for 26.75"w x 48"h right-hinged cabinet

AireCore: SEA1927SGRWVD (glass door), SEA1927SSRWVD (solid door)

AireCore with Brushed Aluminum: SEB1927SGRWVD (glass door), SEB1927SSRWVD (solid door)

Stainless steel: SES1927SGRWVD (glass door), SES1927SSRWVD (solid door)

Door for 26.75"w x 60"h left-hinged cabinet

AireCore: SEA1927SGLBVD (glass door), SEA1927SSLBVD (solid door)

AireCore with Brushed Aluminum: SEB1927SGLBVD (glass door), SEB1927SSLBVD (solid door)

Stainless steel: SES1927SGLBVD (glass door), SES1927SSLBVD (solid door)

Door for 26.75"w x 60"h right-hinged cabinet

AireCore: SEA1927SGRBVD (glass door), SEA1927SSRBVD (solid door)

AireCore with Brushed Aluminum: SEB1927SGRBVD (glass door), SEB1927SSRBVD (solid door)

Stainless steel: SES1927SGRBVD (glass door), SES1927SSRBVD (solid door)

Door for 26.75"w x 92"h left-hinged cabinet

AireCore: SEA1927SGLVD (glass door), SEA1927SSLVD (solid door)

AireCore with Brushed Aluminum: SEB1927SGLVD (glass door), SEB1927SSLVD (solid door)

Stainless steel: SES1927SGLVD (glass door), SES1927SSLVD (solid door)

Door for 26.75"w x 92"h right-hinged cabinet

AireCore: SEA1927SGRVD (glass door); SEA1927SSRVD (solid door)


AireCore with Brushed Aluminum: SEB1927SGRVD (glass door); SEB1927SSRVD (solid door)

Stainless steel: SES1927SGRVD (glass door); SES1927SSRVD (solid door)



Scope Drying and Tracking Retrofit Unit

Product Number Reference

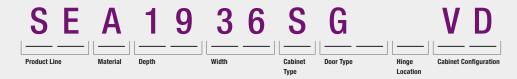

Product Numbers

To order a scope drying and tracking retrofit unit, first specify the replacement door size to match the cabinet size, and then specify the drying and tracking retrofit unit.

Left-hinged door for 36"w cabinet

AireCore: SEA1936SGLVD (glass door); SEA1936SSLVD (solid door)

AireCore with Brushed Aluminum: SEB1936SGLVD (glass door); SEB1936SSLVD (solid door)

Stainless steel: SES1936SGLVD (glass door); SES1936SSLVD (solid door)

Left-hinged door for 40"w cabinet

AireCore: SEA1940SGLVD (glass door); SEA1940SSLVD (solid door)

AireCore with Brushed Aluminum: SEB1940SGLVD (glass door); SEB1940SSLVD (solid door)

 $\textbf{Stainless steel:} \ \textbf{SES1940SGLVD} \ (\textbf{glass door}); \ \textbf{SES1940SSLVD} \ (\textbf{solid door})$



Scope Drying and Tracking Retrofit Unit

Product Number Reference



Product Numbers

Scope Drying and Tracking Retrofit Unit – For 26.75"w left-hinged cabinet

5-port unit: SVDURL5 **10-port unit:** SVDURL10

Scope Drying and Tracking Retrofit Unit – For 26.75"w right-hinged cabinet

5-port unit: SVDURR510-port unit: SVDURR10

Scope Drying and Tracking Retrofit Unit – For 36"w cabinet

8-port unit: SVDUR8 16-port unit: SVDUR16

Scope Drying and Tracking Retrofit Unit – For 40"w cabinet

9-port unit: SVDUR918-port unit: SVDUR18



Scope Drying and Tracking Retrofit Unit

Replace these items every 6 months for optimal performance:

- Compressor tube/air filter replacement, package of 1: SVCTAF
- Triple manifold tube, package of 1: SVTMT1
- . Manifold tube replacement, package of 1: SVMT1
- · Manifold auxiliary water tube replacement, package of 1: SVWAT1
- HEPA fan filter replacement: SHEPFIL (package of 1), SHEPFIL3 (package of 3)

Single-use connector kits:

To find which connector kit is compatible with your endoscope, visit https://www.innerspacehealthcare.com/Ventaire

To order, call an InnerSpace Customer Service representative at 888-435-2256

Materials and Finishes

Compressor:

Powder-coated steel

Manifold:

Powder-coated steel

Tubing:

Soft PVC

Compressor tube/air filter:

Soft PVC tube and polypropylene filter

Connectors (vary by type):

HDPE, nylon (white), polypropylene (white) and polycarbonate (clear)

