


# Mounting a Scope Drying Unit to IV or Scope Pole

Ventaire™ Endoscopy System

Follow these instructions to mount a scope drying unit to an IV or scope pole.

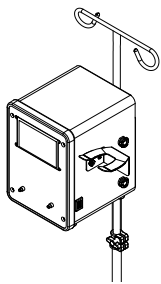

Included:

- Mounting bracket
- 2 U-bolts
- 4 nuts

Tools needed:


- Flat-head screwdriver
- Wrench

## Step 1

Position bracket so the arrow faces up


## Step 2

Place bracket against the pole at desired height. Typical height of unit is approximately 4' 9" (57") from the floor to the center of the unit's screen

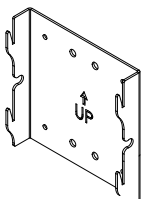


## Step 3

Place U-bolts over the pole and through the center holes in the back of the bracket. Insert and turn the 4 nuts clockwise with wrench until firm and the U-bolts are tightly in place

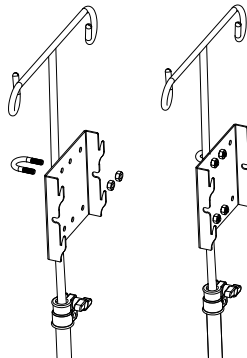

## Step 4

Line up the 4 tabs on the bracket with the 4 channels on the back of the unit. Push down gently on the scope drying unit until firmly in place

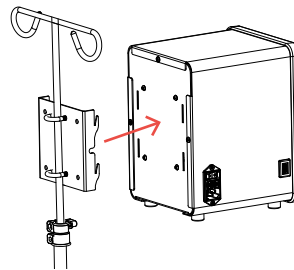

## Step 5

Use the flathead screwdriver to tighten the cam lock on each side of the mounting bracket, making sure the arrow on the cam lock is pointed up or down when the lock is tight

