


Mounting a Scope Drying Unit

Drying Unit to a Wall

Ventaire™ Endoscopy System

Instruction Guide for Scope Drying Unit

Follow these instructions to mount a scope drying unit to a wall.

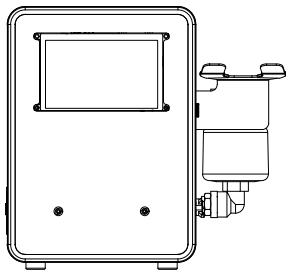

Included:

- Mounting bracket
- 4 wall anchors
- 4 screws
- Mounting template

Note: Use wall anchors when mounting unit to a hollow wall. If mounting to wall studs, you will not need the wall anchors.

Tools needed:

- #8 Phillips head screwdriver
- Drill with 5/16" drill bit

Step 1

Place template at desired location on wall. Typical height of unit is approximately 4' 9" (57") from the floor to the center of the unit's screen

Step 2


Drill 5/16" diameter hole into the 4 locations. Discard template

Step 3

Fold anchor in the middle. Insert anchor in hole and tap until it is flush with wall. Only 3/4" clearance is needed behind wall


Step 4

Position bracket so the arrow faces up and place it over the anchors. Insert screws and tighten with screwdriver until screws are flush with bracket face. Do not overtighten screws

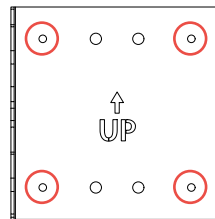


Step 5

Line up the 4 tabs on the bracket with the 4 channels on the back of the unit. Push down gently on the scope drying unit until firmly in place

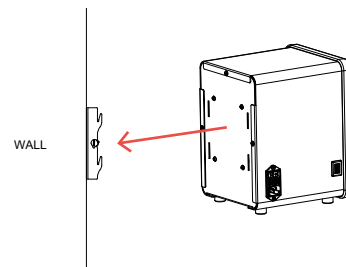

Step 6

Use the flathead screwdriver to tighten the cam lock on each side of the mounting bracket, making sure the arrow on the cam lock is pointed up or down when the lock is tight

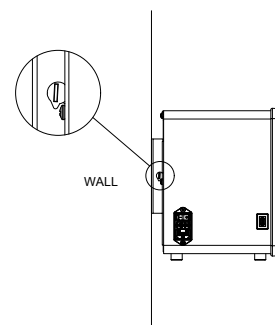

Template for Mounting a Scope Drying Unit to a Wall

Drill 5/16" diameter hole into the 4 locations. Discard template

