

Updating your Ventaire™ Drying and Tracking System Software

Follow these instructions to update the software on your Ventaire cabinet.

1. Download the software update file from innerspacehealthcare.com/software/Ventaire and save it to a USB drive.

Note: The .swu file cannot be placed within a folder and must be the only .swu file in this location

2. Insert the USB drive into the USB port on the back of the Ventaire display unit.

Note: This feature supports FAT12, FAT16, FAT32, and ExFAT USB types. It does not support Windows proprietary USB type NTFS.

3. The update will begin once the USB drive is inserted. The update will take approximately 3 minutes and will automatically restart the system.

4. Remove the USB drive after system restarts.

